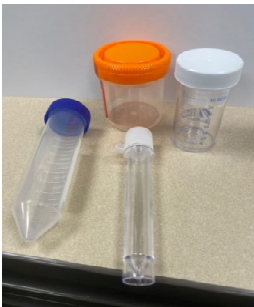
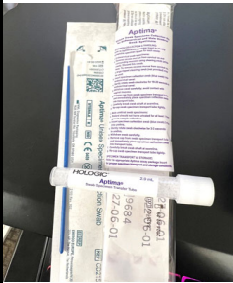
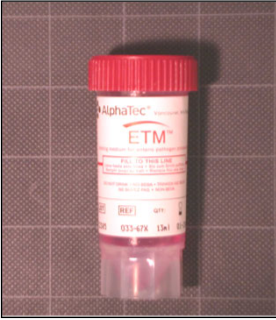
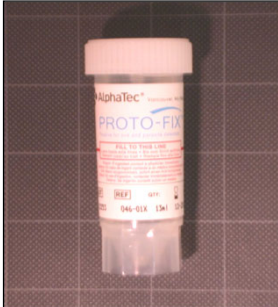



| Device                                                                                                                                               | Recommended Uses                                                                                                                                                      | Acceptable but Not Recommended                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                            | NEVER USED                                                                                                                                                                                                                     | Device name /Item Class & Stock #/Warehouse Description                                                                             |
|------------------------------------------------------------------------------------------------------------------------------------------------------|-----------------------------------------------------------------------------------------------------------------------------------------------------------------------|-----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|--------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|-------------------------------------------------------------------------------------------------------------------------------------|
| Copan flocked E-swab<br>                                             | Aerobic bacterial culture<br>Anaerobic bacterial culture<br>Yeast culture<br>Gram Stain<br>Fungal Stain                                                               | Skin Scrapings, send tissue<br>Mycobacteria (AFB) cultures send unpreserved patient sample or tissue<br>Fungal Culture send unpreserved patient sample or tissue<br>Fluid Culture send unpreserved patient sample<br>Tissue culture send unpreserved patient sample or tissue<br>Operative samples send unpreserved patient sample or tissue<br>Rapid Strep testing use EZ II culturette swab<br>Respiratory Culture<br>Wet Preparation                                                                                                                                                                                                                                                                   | Bacteriology PCR<br>Mycology PCR<br>Viral Culture<br>Stool Culture<br>Chlamydia nucleic acid testing (DNA/RNA)<br>Gonorrhea nucleic acid testing (DNA/RNA)<br>HPV nucleic acid testing<br>Trichomomas nucleic acid testing     | Copan flocked E-swab<br>Item Class: Misc. Patient Care Supplies: 554409<br><b>Description: SWAB COPAN ESWAB 480C</b>                |
| BD EZ II swab<br>                                                   | Rapid Strep testing<br>Confirmatory streptococci culture<br>COVID, Rapid NAAT<br>Influenza A+B NAAT<br>RSV NAAT                                                       | Virus culture, unless immediately placed in M5 buffer<br>Viral PCR unless immediately placed in M5 buffer<br>Mycoplasma PCR unless immediately placed in M5 buffer<br>Ureoplasma PCR unless immediately placed in M5 buffer<br>Aerobic bacterial Culture use Copan flocked E-swab<br>Skin Scrapings, send tissue<br>Mycobacteria (AFB) cultures send unpreserved patient sample or tissue<br>Fungal Culture send unpreserved patient sample or tissue<br>Fluid Culture send unpreserved patient sample<br>Tissue culture send unpreserved patient sample or tissue<br>Operative samples send unpreserved patient sample or tissue<br>Respiratory syncitial Virus antigen<br>Influenza A+B antigen testing | Anaerobic culture<br>Bacteriology PCR<br>Mycology PCR<br>Stool Culture<br>Chlamydia nucleic acid testing (DNA/RNA)<br>Gonorrhea nucleic acid testing (DNA/RNA)<br>HPV nucleic acid testing<br>Trichomomas nucleic acid testing | BD EZ II swab<br>Item Class: Misc. Patient Care Supplies: 554409<br><b>Description: SWAB BD EZII DUAL 220145</b>                    |
| Sterile containers suitable for<br>Aerobic / Anaerobic Culture<br> | Aerobic Culture<br>Anaerobic culture<br>Gram Stain<br>Fungal Stain<br>Mycobacteria (AFB) stain<br>Mycobacteria (AFB) culture<br>Fungal/Yeast Culture<br>Fluid culture | Virus Culture send unpreserved patient sample                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                             | Viral PCR<br>Bacteriology PCR<br>Mycology PCR<br>Wet preparation<br>Cytology<br><br><b>Sterile syringe <i>with</i> Needle</b>                                                                                                  | A.C.T II (Anaerobic transport)<br>Item Class: Laboratory Supplies: 21608<br><b>Description: ACT II Dual Collection System 21608</b> |

| Device                                                                                                                                                                                           | Recommended Uses                                                                                                                                                                                                                                                                                                                                        | Not Recommended                                  | NEVER USED                                                                                                                                                                                                                                                 | Device name /Item Class & Stock #/Warehouse Description                                                                                                                                        |
|--------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|---------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|--------------------------------------------------|------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| Sterile container suitable for Aerobic Culture<br>                                                               | Aerobic Culture<br>Urine Culture<br>Sputum culture<br>Gram Stain<br>Fungal Stain<br>Mycobacteria (AFB) culture<br>Mycobacteria (AFB) stain<br>Fungal/Yeast Culture<br>Wet preparation<br>Legionella antigen testing (Urine)<br>Streptococcus pneumoniae antigen testing (Urine)<br>Legionella culture<br>Cytology<br>Mycobacterial CX<br>Legionella PCR |                                                  | Anaerobic Culture<br><br>Anaerobic Culture, <b><u>ONLY IF</u></b> tissue present                                                                                                                                                                           | Item Class: Laboratory Supplies: 466001<br>Container Specimen 3 oz 466001<br><br>Item Class: Laboratory Supplies: 470106<br>LAB Centrifuge tube 25/pk 470106                                   |
| Clean container<br>                                                                                              | Clostridium difficile<br><br>Wet Preparation<br>Ova and Parasite Other<br><br>Fungal Culture                                                                                                                                                                                                                                                            | Aerobic culture, send unpreserved patient sample | Aerobic culture<br><br>Anaerobic Culture<br>Stool culture                                                                                                                                                                                                  | Item Class: Laboratory Supplies: 466001<br>Container Specimen 3 oz 466001<br><br>Item Class: Laboratory Supplies: 470138/470139<br>Tube Lab Specimen Urine - 12 ml<br>Cap Lab Spec tube 500/ca |
| Hologic Aptima<br><b>Urine</b> Specimen Collection Kit<br><br>Initial sample, <b><u>not</u></b> mid-stream<br> | Chlamydia trachomatis antigen<br>Neisseria gonorrhoeae antigen<br><br>"Nucleic Acid Amplification - RNA"<br><br>In-house Chlamydia, GC testing for urine<br>Quest Trichomonas testing for urine                                                                                                                                                         |                                                  | Sexual Abuse<br>Rape<br>Chlamydia Culture / PCR<br>GC Culture / PCR<br><br>Insufficient Sample<br>Excess Sample<br><br>HPV nucleic acid testing<br><br>Trichomonas nucleic acid testing --> Mayo testing<br><br>Aerobic / Anaerobic / Fungal / AFB culture | Hologic Aptima:<br><b><u>Urine</u></b> Specimen Collection Kit<br><br>Item Class: Laboratory Supplies: 51290<br><br><b>Description: GENPROBE URINE COLLECTION 1040</b>                         |

| Device                                                                                                                                                                                           | Recommended Uses                                                                                                                                                                                                        | Not Recommended | NEVER USED                                                                                                                                                                                         | Device name /Item Class & Stock #/Warehouse Description                                                                                                         |
|--------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|-------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|-----------------|----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|-----------------------------------------------------------------------------------------------------------------------------------------------------------------|
| <b>Hologic Aptima</b><br><b>Unisex</b> Specimen Collection Kit<br><br>Endocervical and Male urethral samples<br> | Chlamydia trachomatis antigen<br>Neisseria gonorrhoeae antigen<br><br>"Nucleic Acid Amplification - RNA"<br><br>Trichomonas nucleic acid testing<br><br>In-house Chlamydia, GC and Trich testing for cervix and urethra |                 | Sexual Abuse<br>Rape<br>Chlamydia Culture / PCR<br><br>GC Culture / PCR<br>White Swab<br><br>White and Blue Swab<br><br>HPV nucleic acid testing<br><br>Aerobic / Anaerobic / Fungal / AFB culture | <b>Hologic Aptima</b><br><b>Unisex</b> Specimen Collection Kit<br><br>Item Class: Laboratory Supplies: 51291<br><b>Description: GENPROBE SPECIMEN SWAB 1041</b> |
| Device                                                                                                                                                                                           | Recommended Uses                                                                                                                                                                                                        | Not Recommended | NEVER USED                                                                                                                                                                                         | Device name /Item Class & Stock #/Warehouse Description                                                                                                         |
| <b>Hologic Aptima</b><br><b>Multitest</b> Specimen Collection Kit<br>                                           | Chlamydia trachomatis antigen Oral or rectal<br>Neisseria gonorrhoeae antigen Oral or Rectal<br>ARUP Vaginitis Panel<br>In-house Chlamydia and GC testing for Vaginal, Throat, and Rectal Swab Specimens                |                 |                                                                                                                                                                                                    | Item Class: Laboratory Supplies: 60627                                                                                                                          |

|                                                                                                                                |                                                                                                                                                                                                              |  |                                                                                                                                 |                                                                                                                     |
|--------------------------------------------------------------------------------------------------------------------------------|--------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|--|---------------------------------------------------------------------------------------------------------------------------------|---------------------------------------------------------------------------------------------------------------------|
| <p>Hologic - The "ThinPrep"<br/>Pap Test</p>  | <p>Chlamydia trachomatis antigen<br/>Neisseria gonorrhoeae antigen</p> <p>HPV antigen<br/>HPV genotyping HPV-16, 18/45</p> <p>Alternative is a green Aptima swab</p> <p>Trichomonas nucleic acid testing</p> |  | <p>Sexual Abuse<br/>Rape<br/>Chlamydia Culture / PCR<br/>GC Culture / PCR</p> <p>Aerobic / Anaerobic / Fungal / AFB culture</p> | <p>Item Class: Laboratory Supplies: 70136<br/>Thin Prep PAP test / 25 per pack</p>                                  |
| <p>Alpha Tek ETM</p>                           | <p>GI Diarrheal Panel</p> <p>Clostridium difficile</p> <p>Ova &amp; Parasite: Cryptosporidium,<br/>Giardia, Cyclospora,<br/>Entamoeba histolytica</p>                                                        |  | <p>Ova and Parasite - Microscopic</p>                                                                                           | <p>Alpha Tek ETM</p> <p>Item Class: Laboratory Supplies: 900612</p> <p>Description: <b>PARASET ETM ALPHATEC</b></p> |

| Device                                                                                                  | Recommended Uses                                                                                                                          | Not Recommended | NEVER USED                                                                                        | Device name /Item Class & Stock #/Warehouse Description                                                                                       |
|---------------------------------------------------------------------------------------------------------|-------------------------------------------------------------------------------------------------------------------------------------------|-----------------|---------------------------------------------------------------------------------------------------|-----------------------------------------------------------------------------------------------------------------------------------------------|
| Alpha Tek PROTO CLR<br> | Ova and Parasite - Microscopic<br>(Contact Microbiology at 3707 to place the order)<br><br>Can use Green ECOFix sent to Mayo Medical labs |                 | Clostridium difficile<br><br>GI Diarrheal Panel<br><br>Aerobic / Anaerobic / Fungal / AFB culture | Alpha Tek PROTO CLR<br><br>Item Class: Laboratory Supplies: 400405<br><br><b>Description: PROTOFIX CLR 4602CS</b>                             |
| Device                                                                                                  | Recommended Uses                                                                                                                          | Not Recommended | NEVER USED                                                                                        | Device name /Item Class & Stock #/Warehouse Description                                                                                       |
|                       | Aerobic Anaerobic Blood Culture                                                                                                           | Fluid Culture   | Fungal Culture<br><br>Mycobacteria culture                                                        | Item Class: Laboratory Supplies: 59268<br>BACTALERT Aerobic SA Blue<br><br>Item Class: Laboratory Supplies: 59269<br>BACTALERT Anaerobic Pink |
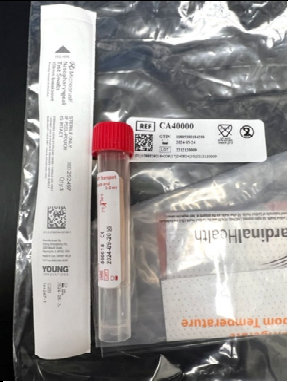
| Device                                                                            | Recommended Uses                                                                                                   | Not Recommended | NEVER USED                               | Device name /Item Class & Stock #/Warehouse Description        |
|-----------------------------------------------------------------------------------|--------------------------------------------------------------------------------------------------------------------|-----------------|------------------------------------------|----------------------------------------------------------------|
|  | Fungal Blood Culture<br>Mycobacteria Blood Culture                                                                 |                 | Aerobic Culture<br><br>Anaerobic culture | Item Class: Laboratory Supplies: 51582<br>Vacutainer Green 6ml |
| Device                                                                            | Recommended Uses                                                                                                   | Not Recommended | NEVER USED                               | Device name /Item Class & Stock #/Warehouse Description        |
|  | Quest<br>COVID Flu Combo testing<br>Respiratory Panel PCR, NP<br><br>Herpes & Varicella DNA<br>probe PCR Viral PCR |                 |                                          | Item Class: Laboratory Supplies                                |